

Technical Information

Clorox Healthcare® **Hydrogen Peroxide Cleaner Disinfectant Wipes**





OVERVIEW

Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes are engineered as ready-touse cleaner-disinfectant wipes for healthcare facilities. They are compatible with a wide range of common healthcare surfaces and have a low residue and odor profile.

These EPA-registered disinfectant wipes contain hydrogen peroxide and other ingredients to kill most bacteria and viruses on a variety of hard, nonporous healthcare surfaces with contact times of 30 seconds or 1 minute.

INGREDIENT INFORMATION

Active ingredient: Hydrogen Peroxide	1.4%
Other ingredients:	98.6%
Total (not including weight of wipe):	100.00%

AVAILABLE FORMATS

Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wines are available in multiple d

Clorox Healthcare® Hydrogen Peroxide **Cleaner Disinfectant Wipes EPA Registration** 67619-25 Number **Product Classification** Pre-moistened wipes **Dilution** Ready to use 1-step **Disinfectant Type** cleaner-disinfectant **Active Ingredient** Hydrogen Peroxide Chemistry Odor Cleaning Agent Expiration date listed on each **Shelf Life** package: month, day, year **Toxicity Signal Word** CAUTION **HMIS Rating** 1-0-0-A

At a Glance

sizes and formats to accomplish different					
disinfecting jobs.	accomplian amerent	Designed for	Item#	Quantity	Shelf Life
Clinical Surface Wipes (6.75" x 5.75")		Daily Disinfection Small high-touch surfaces Medical equipment	30825	155 ct. canister, 6/case	24 months
Multi-Purpose Wipes (6.75" x 9")	•	Daily Disinfection Patient room surfaces	30824	95 ct. canister, 6/case	24 months
(6.75 × 5)		Medical equipment	31426	50 ct. singles, 6/case	24 months
Terminal Wipes		Discharge Disinfection	30826	185 ct. bucket, 2/case	24 months
(12" x 11")		Large areas	30827	185 ct. refill, 2/case	24 months

PPE Required

Follow your facility's protocols



Technical Information

Clorox Healthcare®
Hydrogen Peroxide
Cleaner Disinfectant Wipes





DISINFECTION DIRECTIONS FOR USE#



 REMOVE gross soil if visible.*



WIPE surface with towel until completely wet. Gloves should be worn.



To DISINFECT, let stand for the required contact time as listed on the product back label.

Special Instructions for Use Against HIV-1, HBV and HCV:

This product kills HIV-1, HBV and HCV on pre-cleaned hard, nonporous surfaces/objects previously soiled with blood/body fluids in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus (HIV-1) (associated with AIDS), Human Hepatitis B (HBV) and Human Hepatitis C (HCV).

Special Instructions for Using This Product to Clean and Decontaminate Against HIV-1, HBV and HCV on Surfaces/Objects Soiled with Blood/Body Fluids:

Personal Protection: When handling items soiled with blood or body fluids, use disposable impervious gloves, gowns, masks and eye coverings.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and other objects before applying this product.

Contact Time: Allow surfaces to remain wet for HBV, HCV and HIV-1 for 30 seconds. The contact times for other bacteria, viruses, and fungi may differ. See product label for contact times.

Disposal of Infectious Materials: Use disposable impervious gloves, gowns, masks and eye coverings. Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

^{*} See product label for complete instructions. *This is a one-step product. Precleaning is required only if blood, vomit, fecal matter or other soil is visible.



Technical Information

Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes





DISINFECTION EFFICACY CLAIMS FOR WIPES

	Disinfecting efficacy against:	KILL TIMES
	Acinetobacter baumannii	1 min
	Campylobacter jejuni	1 min
	Enterobacter aerogenes	30 sec
	Vancomycin-resistant Enterococcus faecalis (VRE)	30 sec
	Multidrug-resistant Enterococcus faecium	30 sec
	ESBL-producing Escherichia coli	30 sec
	Escherichia coli	30 sec
	Escherichia coli O157:H7	30 sec
	Escherichia coli (NDM-1)	30 sec
	Klebsiella oxytoca	30 sec
<u>.e</u>	Carbapenem-resistant Klebsiella pneumoniae	30 sec
ter	Klebsiella pneumoniae (NDM-1)	30 sec
Bacteria	ESBL-producing Klebsiella pneumoniae	30 sec
-	Multidrug-resistant Klebsiella pneumoniae	30 sec
	Pseudomonas aeruginosa	30 sec
	Salmonella enterica	30 sec
	Staphylococcus aureus	1 min
	Linezolid-resistant Staphylococcus aureus (LRSA)	1 min
	Methicillin-resistant Staphylococcus aureus	1 min
	Methicillin-resistant Staphylococcus aureus (CA-MRSA USA300)	1 min
	Stenotrophomonas maltophilia	30 sec
	Penicillin-resistant Streptococcus pneumoniae	30 sec
	Streptococcus pyogenes	30 sec
	Avian Influenza A Virus (H5N1)	1 min
	Cytomegalovirus	30 sec
	Hepatitis B Virus (HBV) [†]	30 sec
	Hepatitis C Virus (HCV)	30 sec
	Herpes Simplex Virus Type 1 (HSV-1)	30 sec
	Herpes Simplex Virus Type 2 (HSV-2)	30 sec
es	Human Immunodeficiency Virus Type 1 (HIV-1)	30 sec
Viruses	Human Coronavirus	30 sec
5	Influenza A Virus (H1N1)	30 sec
	Influenza A Virus/Hong Kong	30 sec
	Influenza B Virus	30 sec
	Norovirus [†]	3 min
	Respiratory Syncytial Virus (RSV)	30 sec
	Rhinovirus	1 min
	Rotavirus	1 min
Bi	Candida albicans	5 min
Fungi	Trichophyton mentagrophytes (Athlete's Foot Fungus)	5 min
TB	Mycobacterium bovis (TB)	5 min

 $\label{thm:calicivirus.} \dagger \text{Obtained via a surrogate using feline calicivirus.} \\ \dagger \text{www.osha.gov/SLTC/healthguidelines/isopropylalcohol/recognition.html}$

Use as directed on hard, nonporous surfaces. All claims based on data as of 2/2014.



Technical Information Clorox Healthcare®

Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes





SAFETY INFORMATION

Reference the Safety Data Sheet (SDS) sheet for complete product safety information.

EPA Label Statements

This chemical is a pesticide product registered by the Environmental Protection Agency and is subject to certain labeling requirements under federal pesticide law. These requirements differ from the classification criteria and hazard information required for safety data sheets and for workplace labels of non-pesticide chemicals. Following is the hazard information as required on the pesticide label:

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

FIRST AID: IF IN EYES: Hold eye open and rinse slowly and gently with water for 15–20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or when going for treatment.

PHYSICAL AND CHEMICAL HAZARDS: The product contains no substances which at their given concentration, are considered to be hazardous to health

National Fire Protection Association (NFPA) Ratings

Health Hazard: 1 Flammability: 0 Instability: 0

Physical and Chemical Hazards: None

Questions? Call (800) 234-7700

Hazardous Materials Identification System (HMIS) Ratings

Health Hazard: 1 Flammability: 0 Physical Hazard: 0 Personal Protection: A