





				Bleach			Hydrogen Peroxide	Quat
Вє	eds/Pads	Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
MIDMARK	Medical Exam Bed (Acute & Physicians)							
MIZUHOSI	Surgical Pads							
SKYTRON®	Surgical Compressions Pad, and Surgical Table Disinfecting							
	Hercules 6701 Surgical Table							
	Beds							
	Stretchers							
	Patient Support Surfaces							
STRYKER	Furniture							
	EMS cots							
	Fasteners							
	Model D860 Operating Table							
Hill-Rom	Mattresses *Occasional use only; rinse with tap water after use							
	Envella Air Fluidized Therapy System							
AliMed	AliGel O.R. Products Pads and Positioners					6%		
Curbell Medical	Universal Mattress							
Unisoft	Unisoft One® Mattresses							
Anacom	MedTek Series 7 and 8 Pillow Speaker							Clorox Disinfecting Wipes
Medical Positioning, Inc.	UltraScan Table/GSPM Table/ Rapid Response HUT Table/ HUT Table							
Bella VMS	Waterproof Vet Bed/ Mat for Fold Down Exam Tables							
	Rehab Platform 2 Bariatric Patient Bed Frame	<i>C. Diff</i> Prevention						
Sizewise Bari	Rehab Platform 3 Bariatric Patient Bed Frame	<i>C. Diff</i> Prevention						
Linet								<i>C. Diff</i> Prevention







				Bleach			Hydrogen Peroxide	Quat
Blood G	lucose Meters	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare VersaSure Alcohol-Fre Disinfectant Wipes
Manufacturer	Product							
ARKRAY	ASSURE® Platinum Glucose Meter							
ARRAI	ASSURE® Prism							
Bayer	CONTOUR® Link							
Dayer	CONTOUR® USB Meters							
	OneTouch® Blood Glucose Meters							
	OneTouch® Verio Blood Glu- cose Meters							
Lifescan	OneTouch® VerioFlex Blood Glucose Meters							
	OneTouch® VeriolQ Blood Glucose Meters							
Nova Biomedical	Nova StatStrip Glucose Hospital Meter							
	Nova StatStrip Xpress Glucose Hospital Meter							
Roche Diagnostics	ACCU-CHEK® Inform II Base Unit							
TAIDOC	TaiDoc							
Medline	EVENCARE G2							
ricamic	EVENCARE G3							
	FreeStyle InsuLinx®							
	FreeStyle Lite*, Freedom Lite*							
	Precision Xtra™ Glucose & Ketone							
	FreeStyle Lite Blood Glucose Monitoring System							
	FreeStyle InsuLinx Blood Glucose Monitoring System							
ALL	FreeStyle Libre 2 Blood Glucose Monitoring System							
Abbott	FreeStyle Libre 3 Blood Glucose Monitoring System							
	FreeStyle Libre Blood							
	Glucose Monitoring System FreeStyle Freedom Lite Blood Glucose Monitoring System							
	FreeStyle Optium							
	FreeStyle Optium Neo H							
	FreeStyle Precision Pro							
Medtronic	iPro2 Glucose Monitoring System							







				Bleach			Hydrogen Peroxide	Quat
	ion Pumps	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
BD ALARIS PUMPS (FORMLY CAREFUSION)	CareFusion Alaris* Systems							
HOSPIRA	Omni-Flow 4000* Plus IV Medication Management System							
NOSPIRA	GemStar® Infusion Pumps							
	LifeCare PCA 7 Infusion System							
AGILIA INTUITIVE GENERATION	Volumetric Infusion Pump							
BAXTER	ASSURE* 4							







				Bleach			Hydrogen Peroxide	Quat
Furni	ture/Apparel	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Kwalu	All furniture products							
healtHcentric	IC+ Upholstery Solution™							
Prime Medical	SAF-T™ Scrubs							
Midmark	Upholstery							
Manark	Manual Examination Tables							
DentalEZ Equipment Solutions	JV Generation Chair					1:10		
Graham-Field	LUMEX FR577RG/RGH Series Clinical Care Recliners					1:10		
Clinton Industries	Lab-X Series Padded Blood Drawing Chairs							
Activeaid, Inc.	Model 285/20 Tilt-In-Space Adult Shower/Commode/ Wheel Chair							
Harloff Company, Inc.	Powder Coated and Stainless Steel Carts					0.05%		Clorox Disinfecting Wipes
Wheelchairs of Kansas	Model 38060000 Bariatric Lift and Transfer Unit							
A-Dec	Dental Chair Upholstery							







				Bleach			Hydrogen Peroxide	Quat
Pr	und/Transducer obes (GE)	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare [®] VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
	LOGIQ™ TRANSDUCERS/ ULTRASOUNDS 10L, 10S							
	10LB, 10LB-RS 10S-RS 11L, 11L-D							
	12L 12L-RS, 12L-SC 12S-D							
	12S-RS 16-RS							
	16L-RS 3.5C, 3.5CS, 3.5 gray lens, 3C, 3Cb, 3CRF, 3CRF-D, 3C-RS, 3S, 3SC-RS, 3S-D, 3S-LC, 3SR, 3S-RC, 3S-RS, 3S-SC,							
	3V, 3V-D 4C 4C-A, 4C-D							
	4C-RS, 4C-SC 4D10L 4D16L, 4D3C-L, 4D5C-L,							
General Electric	4DE7C, 4S, 4V-D 4Vc-D 4D8C							
	546L-V 5CS 5C, 5S, 5S-LC,							
	5S-RS 6S, 6S-D							
	6S-RS 739L 7L, 7S, 7S-R5							
	8C, 8C-RS 8L 8L-RS							
	9L, 9L-D, 9L-RS AB2-5, AB2-7, AB2-7-D, AB2- 7-RS, AB4-8							
	AC2-5 BE9C, BE9CS, BE9CS-D, BE9CS-RS							
	C1-5-D C1-5-RS, C1-6-D, C1-6VN-D							

				Bleach			Hydrogen Peroxide	Quat
	und/Transducer obes (GE)	HEATTCASE THE TOTAL CONTROL OF THE TOTAL CONTROL O		FILTON CO.	THE OF TRANSPORT	total	TELEPLANE TOWN THE WAY TO SERVICE AND THE SER	A STATE OF THE STA
	(cont.)	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
	C31, C3-10-D, C55, C551, CAE, CS							
	C2-5-RC, C2-9-D, C2-9VN-D							
	C3-10-D, C4-8-D							
	C358, C36, C364, C386, C721, CFB, CZB							
	Dual Probe Vscan							
	ERB							
	E72, E721 E8C, E8C-RS, E8CS, E8CS-RC, E8Cs-RS, E8CS-SC							
	eM6C eM6C G2							
	G3C Probe on Vscan, C3S							
	Probe on Vscan							
	i12L, i12L-RS							
	IC5-9, IC5-9W-RS							
	I739, I739-RS IC5-9-D, IC5-9H, IC9-RS							
	L39, L76, L764, LB, LD, LE, LH, LI, LT							
	LA39 L12n-SC, L3-12-D, L4-12t-RS, L6-12-RC, L6-12-RS, L8-18i-D,							
	L8-18i-RS, L8-18i-SC L12n-RS							
General Electric	M12L, M12L-H, M3S, M4S, M4S-D, M4S-RS, M5Sc-D, M5Sc-RS, M6C, M7C, M7C-H, ML6-15, ML6-15-D, ML6-15-RS,							
	MTZ, MZ							
	M3-8							
	PA2-5P PCW4.0							
	RAB2-5, RAB2-5-D, RAB2-5-RS, RAB2-5L, RAB-2-6-RS, RAB4-8-D, RAB4-8-RS, RAB4-8L, RAB4-8P, RAB6-D, RAB6-RS, RIC5-9, RIC5-9-D,							
	RIC5-9-RS, RIC5-9A-RS, RIC5-9H, RIC5-9W, RIC5-9W-RS, RIC6-12-D, RRE5-10-D, RSP5-12, RSP6-12, RSP6-16, RSP6-16-D,							
	RSP6-16-RS RNA5-9, RNA5-9-D, RNA5-9-RS							
	RAM3-8, RM14L, RM6C, RRE6-10, RRE6-10-D, RSM5-14							
	S222, S317, S611							
	S220, S316, SCW2.0							
	SP10-16-RS, SP4-10, SP6-12							
	T739, T739-RS LogiQ S8					1:10 of 5%	Wipes	
	E0310: 30			<u> </u>		1.10 01 370	wipes	







				Bleach			Hydrogen Peroxide	Quat
	und/Transducer es (Phillips)	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with	Clorox* Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant
		·			Bleach		Disinfectant Wipes	Wipes
Manufacturer	Product							
	ULTRASOUND TRANSDUCERS & PROBES (SYSTEM NAMES):							
	3D6-2 (HD11), 3D8-4, 3D9-3v (HD11, HD15, iU22) ,		T/C	т/с	T/C		T/C	
	7.5MHZ Endo		T/C	T/C	T/C		T/C	
	C5-1 (CX50, HD15, iE33, iU22), C5-2 (HD5, HD6, HD7, HD11, HD15, iE33, iU22), C6-2 Compact (CX30, Sparq), C6-3, C8-4v, C8-5, C8-5 Compact (CX30, CX50),		т/с	т/с	т/с		т/с	
	C9-3io , C9-4, C9-4ec, C9-5ec		T/C	T/C	T/C		T/C	
	L8-4, L9-3, L11-3, L12-3 (HD5, HD6, HD7, HD11, HD15), L12-4 Compact (CX30), L12-5 50 Compact (CX50), L12-5 50 mm, L15-7io (HD11, HD15, iE33, iU22), L17-5		т/с	т/с	т/с		т/с	
Phillips	S3-1, S4-1 (iU22), S4-2, S4-2 Compact (CX30, Sparq), S5-1 (CX50, iE33, iU22), S5-2, S8, S8-3, S8-3 Compact (CX30, CX50, HD5), S12, S12-4, S12-4 Compact (CX50)		т/с	т/с	т/с		т/с	
	V6-2 (HD11XE, HD15, iU22), V8-4		T/C	т/с	T/C		T/C	
	X3-1, X5-1, X7-2 (iE33, iU22)		T/C	T/C	T/C		T/C	
	3D6-2 (iU22), 3D9-3v (Affiniti, ClearVue, EPIQ)		T/C	т/с	T/C		T/C	
	BP10-5ec		T/C	T/C	T/C		T/C	
	C5-1 (Affiniti, EPIQ), C5-2 - ClearVue, C6-2 Compact (Affiniti), C8-4v, C8-5 Compact (Affiniti, EPIQ)		T/C	т/с	т/с		т/с	
	C9-2, C9-3v, C9-4v, C9-4v Compact, C10-3v, C10-4ec		т/с	т/с	T/C		т/с	
	L12-3 (Affiniti, CX50, EPIQ), L12-4 X7-2 (EPIQ), L12-4 Compact (Affiniti), L12-5 50 Compact (Affiniti, EPIQ), L15-7io (Affiniti, CX30, CX50, EPIQ), L15-7io (HD11, HD15, iE33, iU22), L18-5		T/C	т/с	т/с		т/с	

				Bleach			Hydrogen Peroxide	Quat
Prob	ind/Transducer es (Phillips) (cont.)	Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product						711,200	
	S4-1 (ClearVue), S4-2 Compact (Affiniti), S5-1 (Affiniti, EPIQ), S8-3 Compact (Affiniti, EPIQ), S12-4 Compact (Affiniti, EPIQ)		т/с	т/с	т/с		т/с	
	V6-2 (Affiniti, ClearVue, EPIQ, HD5), VL13-5, VL13-5 Compact		T/C	T/C	T/C		T/C	
	X5-1 (EPIQ), X6-1		T/C	T/C	T/C		T/C	
	1.9MHz CW Pencil, 1.9MHz TCD		T/C	т/с	T/C		т/с	
Phillips	5.0MHz CW Pencil		T/C	T/C	T/C		T/C	
	D2cwc, D2tcd, D5cwc		T/C	T/C	T/C		T/C	
	TCD		T/C	T/C	T/C		T/C	
	S6-2mpt S7-3t, S7-2t Omni, S8-3t		С	С	С		т/с/н	
	OMNI III TEE		С	С	С		T/C/H	
	X7-2t		С	С	С		T/C/H	
	L10-4lap		T/C	T/C	T/C		T/C/H	
	15-6L		T/C	T/C	T/C		С	
CO2 Transducer	M1460A							
	Avalon Fetal Transducer							

Legend

Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

- **C** = Transducer
- **H** = Cable and Connector
- T = Handle and Steering Mechanism







				Bleach			Hydrogen Peroxide	Quat
	Ultrasound/Transducer Probes (Other)		Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product		ı			ı		<u>'</u>
	ULTRASOUND TRANSDUCERS & PROBES (SYSTEM NAMES):							
	Acuson CV70 , Acuson SC2000 , Acuson Cypress , Acuson X500 , Acuson X300, Acuson Antares , Acuson X150, and Sonoline G20 UltraSound Systems							
	3V2c, 4V1c, 5.0L45, 5.0P10, 6C1HD, 7.5L75S, 8V3, 10V4							
	BE9-4							
Siemens	C6-2, C7F2, C8-5, CW2, CW5, C5-2, C5F1, CH4-1, CH6-2, CX5-2							
	EC9-4, EV9-4							
	L10-5							
	MPT7-4							
	P4-2, P9-4, P10-4, PH4-1, PX4-1							
	AUX CW							
	VF13-5-SP, VF7-3, VFX13-5, V5Ms, VF13-5-SP, V7M							
	SYSTEM AND MINI-DOCK							
	Edge™							
	S Series™							
	M-Turbo® M-OB/GYN Office™							
	MicroMaxx*							
	TITAN®							
FUJIFILM	Sonosite® 180PLUS™							
SONOSITE, INC.	iLook*15, iLook*25							
	NANOSONICS TROPHON EPR							
	C11, C11e, C11x, C11 (Cast Urethane), C15 (Cast Urethane)							
	C11n							
	C15, C15e							
	C60							

				Bleach			Hydrogen Peroxide	Quat
	und/Transducer oes (Other) (cont.)	Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
	C60e, C60x, C60n							
	D2, D2x							
	HFL38, HFL38x							
	HFL50x							
	L25, L25e, L25x							
	L25n							
	L38, L38e, L38x, L52, L52e (Vet Only)							
	L38n, L38xi, L52x (Vet Only), L52n (Vet Only)							
FUJIFILM	P10, P10x, P17, P21x							
SONOSITE, INC.	P21n							
	C8, C8e, C8x							
	ICT, ICTx							
	ICTn							
	SLA, SLAx, SLT							
	ACCESSORIES							
	NanoMaxx® Dock							
	PowerPack							
	Power Supply P06936, P08792, P09823, P09714							
Welch Allyn	SureTemp® Therms and Probes							
	Transducers							
Toshiba	Diagnostic Ultrasound System Aplio i900, i800 and i700							
Echosens	FibroScan 502/530 Touch Ultrasound and Probes							







				Bleach			Hydrogen Peroxide	Quat
Keybo	ards/Phones/ Cases	Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Whitney Medical Solutions	eShield* Screen Protectors and Cases							
KeySource International	San-a-Key® Keyboards							
Man and Machine	Keyboards and Mice							
Otterbox	Phone and iPad® cases							
Seal Shield	Keyboards and Mice							
Seal Shield	Clean Seal™ Screen Protectors							







				Bleach			Hydrogen Peroxide	Quat
	Lights	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Skytron	Precision HD Handle Camera System							
_	OR Lights							
	Exam Lighting							
Welch Allyn	Green Series™ 300 General Exam Lights							
	ProXenon 350 Surgical Illuminator					1:10		
Trumpf	iLED 5/TruLight 5000 Lighting System							
Sunoptic Technologies	Surgical S400T Titan 400HP/ TITAN 180 Xenon Illuminator							







				Bleach			Hydrogen Peroxide	Quat
Vital Sign Monitors/ Consoles		Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Barco	Eonis® Diagnostic Displays							
	Transport Pro							
GE Healthcare	ApexPro Telemetry System					1:10-1:100		
	ApexPro Antenna Infrastructure, Transmitter and Receiver					1:10-1:100		
Midmark	IQvitals							
Midiliark	IQvitals® Blood Pressure Cuffs							
	IntelliVue Patient Mx400/450/500/ 600/700/800							
Phillips	IntelliVue Telemetry Transmitters							
	IntelliVue Monitors							
	Avalon Fetal Monitors							
	Connex Vital Signs							
	Connex Integrated Wall System					1:10		
	Accessory Cable Management Mobile Stand 4800/4900 Series							
	CareTemp Touch Free Thermometer					1:10		
Welch Allyn	Propaq Encore® Blood Pressure Cuffs							
	Harvey™ Elite® Stethoscopes							
	LumiView™ Binocular Microscope							
	Digital MacroView™ Oto- scopes and Halogen HPX* Illuminators							
MindRay	Accutorr 7 Vital Signs Monitor							
	Passport 12/Passport 8 Patient Monitor							
Mortara Instru- ment, Inc.	ELI 280/150c/250c 12-lead Resting Electrocardiograph					1:500 to 1:10		
Capsule Tech	SmartLinx Vitals Plus					10%		
Exergen	TAT5000 Temporal Thermometer							







	Bleach						Hydrogen Peroxide	Quat
Radiology		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
	Computed Radiology Cassettes							
AGFA Healthcare	DR Detectors (Dx-D 10C, 10G, 20C, 20G)							
	DX-D 100 Digital Radiography X-Ray System							
Hologic	GemStar® Infusion Pumps							
InFab	Aprons and Gloves							
GE Healthcare	Optima XR220amx X-Ray System					5250-6150 ppm		
	Senographe Pristina Mammography System							
Konica Minolta	CS-1 Regius							







		Bleach					Hydrogen Peroxide	Quat
Ventilator Systems		Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare* Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
	Puritan Bennett							
	540 series (409 was OK'd for touchscreen and arm)							
Covidien	840 series (409 was OK'd for touchscreen and arm)							
	980 series (409 was OK'd for touchscreen and arm)							
Kendall SCD	700 Sequential Compression System							
AV Impluse	6060 Controller and Accessories							
Phillips	Triology 100 System							
	Respironics V60/Plus Ventilato					5%; 1:10		
Getinge-Maquet	SERVO i/s Ventilator System and Tube							







				Bleach			Hydrogen Peroxide	Quat
He	althcare IT	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Enovate Medical	Computer Carts (EMC)							
RICOH USA	Interactive Whiteboard							
Panasonic	TOUGHBOOK® 20, 33, 55, A3, G1, L1, N1, T1							
GETT	Cleankeys* (wired and wireless), InduSense*							
	Mobile Cart Computing Solution Venus: 243/223/222/192							
	Fanless Slim HIT Panel PC Solution ONYX- BE182/1723/1731/ 1531/1222/1022							
Onyx Healthcare	Patient Monitor & Smart View Medical Solution ZEUS- 248/228/198							
Ollyx Healthcare	Medical All-In-One Solution AIO-MATE-2403/2203/1903							
	Healthcare Infotainment Solution ONYX-BE321/BE182/1531/ 1222 MedPC- 2100/1600							
	Mobile Medical Assistant MD116/MD101							
	Medical Display Series Solution MEDDP- 524/522/519/518/ 517/515							
НР	Healthcare Edition Notebook/ All in One							
Stanley Healthcare	Hugs and Kisses Tags							
GE Healthcare	Carescape Central Station MP100					5.25%; 1:10 dilution		
TouchPoint InterMetro Industries Corp.	Metro 1800 Series Mobile Workstation							
Capsule Tech	SmartLinx Axon					10%		Clorox Disinfecting Wipes
	SmartLinx Device Identification Module					10%		
Howard Medical	E Series Computerized Work Station							





Clorox Healthcare® Equipment Compatibility Guide

				Bleach			Hydrogen Peroxide	Quat
	Other	Clorox Healthcare* Bleach Germicidal Wipes	Clorox Healthcare Bleach Germicidal Cleaners	Clorox Healthcare* Fuzion Cleaner Disinfectant	Dispatch* Hospital Cleaner Disinfectant Towels with Bleach	Clorox* Germicidal Bleach	Clorox Healthcare Hydrogen Peroxide Cleaner Disinfectant Wipes	VersaSure Alcohol-Free Disinfectant
Manufacturer	Product							
Blickman	Surgical Steel							
Carefusion	Pyxis® Supply Systems							
	Cobas® Urisys 1100®							
Roche	Urine Analyzers							
Sechrist	Chambers							
RF Surgical								
Systems	RF Assure Detection System							
Medline	BioCon 750 Bladder Scanner							
	LIFEPAK 15 Monitor/							
PhysioControl	Defibrillator							
Wolch Allun	Suresight® Vision Screener and					1:10		
WelchAllyn	Autorefractor					1:10		
GE Healthcare	Giraffe OmniBed Carestation					0.50%		
	CS1					0.50%		
Custom	83-S6 Preprocessing Sink					1:32		
Ultrasonics	·					1.32		
AirClean	AC600 Series Laminar Flow					1:10		
Systems	Workstation							
Gentec	882VR Continuous Suction							
	Regulators							
Tomey USA	Pachymeter Probe							
LivaNova Sorin Group	T Heater-Cooler System							
Dwyer Precision Products	BreathCall Series Mounting							Broad Spectrum
Chart Industries	AirSep QuietLife 5							
	EliteOxygen Concentrator							
AirSep	NewLife EliteOxygen Concentrator							
Germfree	BVBI Series/BZ Series Vertical Laminar Flow Workstation					5%		
Germinee	VersaFlow Compounding Aseptic Isolators					<1000 ppm		
Besmay Health-	Curaplex Oxygen Nipple							
care, Inc.	Connector					0.5%		
Zimmer Biomet								
Dornoch Medical	UL-EV100 Transposal Ultra					<1200 ppm		
Systems, Inc.	Evacuation Unit							
Covaris	cryoPREP Extraction System					10%		
	2008K series							
Fresenius Medical Care	2008T series							
Medical Care	Granuflo Concentrate Unit 1							· ·
Draeger	Perseus A500 seris							
APA™	Video Laryngoscope							
Prestan Products	Professional Infant CPR							
	Training Manikins							
Stanley Healthcare	"Hugs* and Kisses* Tags"							