

How to Use Bleach Germicidal Cleaner Spray



1. REMOVE
gross soil.



2. SPRAY
6-8 inches from
surface, until surface
is thoroughly wet.



3. WAIT
before wiping.
Wait 1 minute
for general
disinfection,
3 minutes for
C. difficile spores.



4. WIPE
surface with a
clean damp cloth*
or paper towel to
prevent residue
build up.

Clorox Healthcare®
Bleach Germicidal Cleaner
Item # 68970, 32 fl. oz. Trigger Spray

*Cloths can include:
100% polyester synthetic dry
wipes/ clean "microfiber",
cotton cloths or paper towels

Best Practices for Using Spray Disinfectants



- ▶ Spray disinfectant directly onto the surface. Do not pour into a bucket and apply with cloth or wipes.



- ▶ Apply enough disinfectant to ensure the surface stays wet for the contact time.



- ▶ Make sure the surface stays wet for the contact time listed on the label.



- ▶ Staff who might be sensitive to sprays can wear a mask.



- Refill Instructions:
- ▶ Remove trigger sprayer from empty bottle.
 - ▶ Unscrew cap on refill and pour contents directly into empty bottle.
 - ▶ Use a funnel to minimize product spills.
 - ▶ Replace trigger sprayer and use as directed.



HEALTHCARE™

FROM CLOROXPRO™

For product resources and implementation tools, contact your Clorox sales representative or Call: 1-800-234-7700
Visit: www.CloroxHealthcare.com
Email: healthcare@clorox.com

© 2020 Clorox Professional Products Company,
1221 Broadway, Oakland, CA 94612.
NI-51452