





| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|------------------------------|---|--|---|--|--|---------------------------------|---|---|
| В | eds/Pads | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare® Bleach Germicidal Cleaners | Clorox Healthcare® Fuzion Cleaner Disinfectant | Dispatch® Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| MIDMARK | Medical Exam Bed (Acute & Physicians) | | | | | | | |
| MIZUHOSI | Surgical Pads | | | | | | | |
| SKYTRON® | Surgical Compressions Pad, and Surgical Table Disinfecting | | | | | | | |
| | Hercules 6701 Surgical Table | | | | | | | |
| | Beds | | | | | | | |
| | Stretchers | | | | | | | |
| | Patient Support Surfaces | | | | | | | |
| STRYKER | Furniture | | | | | | | |
| | EMS cots | | | | | | | |
| | Fasteners | | | | | | | |
| | Model D860 Operating Table | | | | | | | |
| Hill-Rom | Mattresses *Occasional use only; rinse with tap water after use | | | | | | | |
| | Envella Air Fluidized Therapy System | | | | | | | |
| AliMed | AliGel O.R. Products Pads and Positioners | | | | | 6% | | |
| Curbell Medical | Universal Mattress | | | | | | | |
| Anacom | MedTek Series 7 and 8 Pillow Speaker | | | | | | | Clorox Disinfecting Wipes |
| Medical Positioning, Inc. | UltraScan Table/GSPM Table/ Rapid Response HUT Table/ HUT Table | | | | | | | |
| Bella VMS | Waterproof Vet Bed/ Mat for Fold Down Exam Tables | | | | | | | |



Clorox Healthcare® Equipment Compatibility Guide



| | | | | | Hydrogen Peroxide | Quat | | |
|----------------------|---|--|---|---|--|---------------------------------|---|---|
| Blood G | ilucose Meters | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | FreeStyle InsuLinx® | | | | | | | |
| Infab | FreeStyle Lite®, Freedom Lite® | | | | | | | |
| Intab | i-STAT® System | | | | | | | |
| | Precision Xtra™ Glucose & Ketone | | | | | | | |
| | ASSURE® 4 | | | | | | | |
| ARKRAY | ASSURE® Platinum Glucose Meter | | | | | | | |
| | ASSURE® Prism | | | | | | | |
| | ASSURE® Pro | | | | | | | |
| Davies: | CONTOUR® Link | | | | | | | |
| Bayer | CONTOUR® USB Meters | | | | | | | |
| EPS Biotech | EasyMax® Glucose Meter | | | | | | | |
| Lifescan | OneTouch® Blood Glucose Meters | | | | | | | |
| Nova Biomedical | Nova StatStrip/StatSensor/ Stat Xpress Glucose Hospital Meter | | | | | | | |
| | ACCU-CHEK® Advantage Meter System | | | | | | | |
| Roche Diagnostics | ACCU-CHEK® Inform Blood Glucose Monitor | | | | | | | |
| | ACCU-CHEK* Inform II Base Unit | | | | | | | |
| Madiina | EVENCARE G2 | | | | | | | |
| Medline | EVENCARE G3 | | | | | | | |
| Abbott | FreeStyle Lite and InsuLinx Blood Glucose Monitoring System | | | | | | | |

Legend







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|---|--|--|---|--|--|---------------------------------|---|---|
| Infus | ion Pumps | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| BD ALARIS PUMPS (FORMLY CAREFUSION) | CareFusion Alaris® Systems | | | | | | | |
| HOSPIRA | Omni-Flow 4000° Plus IV Medication Management System | | | | | | | |
| HOSPIKA | GemStar® Infusion Pumps | | | | | | | |
| AGILIA INTUI- TIVE GENERA- TION | Volumetric Infusion Pump | | | | | | | |
| BAXTER | ASSURE® 4 | | | | | | | |







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|------------------------------------|---|--|---|---|--|---------------------------------|---|---|
| Furni | ture/Apparel | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| Kwalu | All furniture products | | | | | | | |
| healtHcentric | IC+ Upholstery Solution™ | | | | | | | |
| Prime Medical | SAF-T™ Scrubs | | | | | | | |
| Midmark | Midmark* + Ritter* Chairs, Exam Tables + Seating (Up- holstery) | | | | | 1:10 | | |
| Mullark | Synthesis* Cabinetry + Midmark* Workstations | | | | | 1:10 | | |
| DentalEZ Equipment Solutions | JV Generation Chair | | | | | 1:10 | | |
| Graham-Field | LUMEX FR577RG/RGH Series Clinical Care Recliners | | | | | 1:10 | | |
| Clinton Industries | Lab-X Series Padded Blood Drawing Chairs | | | | | | | |
| Activeaid, Inc. | Model 285/20 Tilt-In-Space Adult Shower/Commode/ Wheel Chair | | | | | | | |
| Harloff Company, Inc. | Powder Coated and Stainless Steel Carts | | | | | 0.05% | | Clorox Disinfecting Wipes |
| Wheelchairs of Kansas | Model 38060000 Bariatric Lift and Transfer Unit | | | | | | | |
| A-Dec | Dental Chair Upholstery | | | | | | | |







| | | | | Blooch | | | Hydrogen | Ourt |
|------------------|---|--|---|--|--|---------------------------------|---|---|
| | | | | Bleach | 1 | | Peroxide | Quat |
| | Ultrasound/Transducer Probes (GE) | | Clorox Healthcare® Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | LOGIQ™ TRANSDUCERS/ ULTRASOUNDS | | | | | | | |
| | 10L, 10S | | | | | | | |
| | 10LB, 10LB-RS | | | | | | | |
| | 10S-RS | | | | | | | |
| | 11L, 11L-D | | | | | | | |
| | 12L | | | | | | | |
| | 12L-RS, 12L-SC | | | | | | | |
| | 12S-D | | | | | | | |
| | 12S-RS | | | | | | | |
| | 16-RS | | | | | | | |
| | 16L-RS | | | | | | | |
| | 3.5C, 3.5CS, 3.5 gray lens, 3C, 3Cb, 3CRF, 3CRF-D, 3C-RS, 3S, 3SC-RS, 3S-D, 3S-LC, 3SR, 3S-RC, 3S-RS, 3S-SC, | | | | | | | |
| | 3V, 3V-D | | | | | | | |
| | 4C | | | | | | | |
| General Electric | 4C-A, 4C-D | | | | | | | |
| | 4C-RS, 4C-SC | | | | | | | |
| | 4D10L | | | | | | | |
| | 4D16L, 4D3C-L, 4D5C-L, 4DE7C, 4S, 4V-D | | | | | | | |
| | 4Vc-D | | | | | | | |
| | 4D8C | | | | | | | |
| | 546L-V | | | | | | | |
| | 5CS | | | | | | | |
| | 5C, 5S, 5S-LC, | | | | | | | |
| | 5S-RS | | | | | | | |
| | 6S, 6S-D | | | | | | | |
| | 6S-RS 739L-V | | | | | | | |
| | | | | | | | | |
| | 7L, 7S, 7S-R5 | | | | | | | |
| | 8C, 8C-RS 8L | | | | | | | |
| | 8L-RS | | | | | | | |
| | J = | | | T. Control of the Con | | ı | | |







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|------------------|--|--|---|--|--|---------------------------------|---|---|
| | und/Transducer obes (GE) | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch® Hospital Cleaner Disinfectant Towels with Bleach | Clorox® Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | 9L, 9L-D, 9L-RS | | | | | | | |
| | AB2-5, AB2-7, AB2-7-D, AB2-7-RS, AB4-8 | | | | | | | |
| | AC2-5 | | | | | | | |
| | BE9C, BE9CS, BE9CS-D, BE9CS-RS | | | | | | | |
| | C1-5-D | | | | | | | |
| | C1-5-RS, C1-6-D, C1-6VN-D | | | | | | | |
| | C31, C3-10-D, C55, C551, CAE, CS | | | | | | | |
| | C2-5-RC, C2-9-D, C2-9VN-D | | | | | | | |
| | C3-10-D, C4-8-D | | | | | | | |
| | C358, C36, C364, C386, C721, CFB, CZB | | | | | | | |
| | Dual Probe Vscan | | | | | | | |
| | ERB | | | | | | | |
| | E72, E721 | | | | | | | |
| General Electric | E8C, E8C-RS, E8CS, E8CS-RC, E8Cs-RS, E8CS-SC | | | | | | | |
| | eM6C | | | | | | | |
| | eM6C G2 | | | | | | | |
| | G3C Probe on Vscan, C3S Probe on Vscan | | | | | | | |
| | i12L, i12L-RS | | | | | | | |
| | IC5-9, IC5-9W-RS | | | | | | | |
| | 1739, 1739-RS | | | | | | | |
| | IC5-9-D, IC5-9H, IC9-RS | | | | | | | |
| | L39, L76, L764, LB, LD, LE, LH, LI, LT | | | | | | | |
| | LA39 | | | | | | | |
| | L12n-SC, L3-12-D, L4-12t-RS, L6-12-RC, L6-12-RS, L8-18i-D, L8-18i-RS, L8-18i-SC | | | | | | | |
| | L12n-RS | | | | | | | |
| | M12L, M12L-H, M3S, M4S, M4S-D, M4S-RS, M5Sc-D, M5Sc-RS, M6C, M7C, M7C-H, ML6-15, ML6-15-D, ML6-15-RS, | | | | | | | |







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|------------------|--|--|---|--|--|---------------------------------|---|---|
| | und/Transducer obes (GE) | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | MTZ, MZ | | | | | | | |
| | M3-8 | | | | | | | |
| | PA2-5P | | | | | | | |
| | PCW4.0 | | | | | | | |
| General Electric | RAB2-5, RAB2-5-D, RAB2-5-RS, RAB2-5-L, RAB-2-6-RS, RAB4-8-D, RAB4-8-RS, RAB4-8L, RAB4-8P, RAB6-D, RAB6-RS, RIC5-9-RS, RIC5-9-D, RIC5-9-RS, RIC5-9-RS, RIC5-9H, RIC5-9W, RIC5-9W-RS, RIC6-12-D, RRE5-10-D, RSP5-12, RSP6-12, RSP6-16, RSP6-16-D, RSP6-16-RS | | | | | | | |
| | RNA5-9, RNA5-9-D, RNA5- 9-RS | | | | | | | |
| | RAM3-8, RM14L, RM6C, RRE6- 10, RRE6-10-D, RSM5-14, | | | | | | | |
| | S222, S317, S611 | | | | | | | |
| | S220, S316, SCW2.0 | | | | | | | |
| | SP10-16-RS, SP4-10, SP6-12 | | | | | | | |
| | T739, T739-RS | | | | | | | |
| | Vscan Access convex, Vscan dual (sector/linear) | | | | | | | |
| | Vscan sector | | | | | | | |
| | LogiQ S8 | | | | | 1:10 of 5% | Wipes | |







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|--------------|---|--|---|--|--|---------------------------------|---|---|
| | Ultrasound/Transducer Probes (Phillips) | | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | ULTRASOUND TRANSDUCERS & PROBES (SYSTEM NAMES): | | | | | | | |
| | 3D6-2 (HD11), 3D8-4, 3D9-3v (HD11, HD15, iU22) , | | T/C | T/C | T/C | | T/C | |
| | 7.5MHZ Endo | | T/C | T/C | T/C | | T/C | |
| | C5-1 (CX50, HD15, iE33, iU22), C5-2 (HD5, HD6, HD7, HD11, HD15, iE33, iU22), C6-2 Compact (CX30, Sparq), C6-3, C8-4v, C8-5, C8-5 Compact (CX30, CX50), | | т/с | т/с | т/с | | т/с | |
| | C9-3io , C9-4, C9-4ec, C9- 5ec | | T/C | T/C | T/C | | T/C | |
| Phillips | L8-4, L9-3, L11-3, L12-3 (HD5, HD6, HD7, HD11, HD15), L12-4 Compact (CX30), L12-5 50 Compact (CX50), L12-5 50 mm, L15-7io (HD11, HD15, iE33, iU22), L17-5 | | т/с | т/с | т/с | | т/с | |
| | S3-1, S4-1 (iU22), S4-2, S4-2 Compact (CX30, Sparq), S5-1 (CX50, iE33, iU22), S5-2, S8, S8-3, S8-3 Compact (CX30, CX50, HD5), S12, S12-4, S12-4 Compact (CX50) | | т/с | т/с | т/с | | т/с | |
| | V6-2 (HD11XE, HD15, iU22), V8-4 | | T/C | T/C | T/C | | T/C | |
| | X3-1, X5-1, X7-2 (iE33, iU22) | | T/C | T/C | T/C | | T/C | |
| | 3D6-2 (iU22), 3D9-3v (Affiniti, ClearVue, EPIQ) | | T/C | T/C | T/C | | T/C | |
| | BP10-5ec | | T/C | T/C | T/C | | T/C | |
| | C5-1 (Affiniti, EPIQ), C5-2 - ClearVue, C6-2 Compact (Affiniti), C8-4v, C8-5 Com- pact (Affiniti, EPIQ), | | T/C | T/C | T/C | | т/с | |

- **C** = Cable, connector, and strain reliefs
- **H** = Handle and steering mechanism
- T = Transducer and flexible shaft







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|----------------|---|--|---|--|--|---------------------------------|---|---|
| | ınd/Transducer es (Phillips) | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | C9-2, C9-3v, C9-4v, C9-4v Compact, C10-3v, C10-4ec | | T/C | T/C | T/C | | T/C | |
| | L12-3 (Affiniti, CX50, EPIQ), L12-4 X7-2 (EPIQ), L12-4 Compact (Affiniti), L12-5 50 Compact (Affiniti, EPIQ), L15-7io (Affiniti, CX30, CX50, EPIQ), L15-7io (HD11, HD15, iE33, iU22), L18-5 | | T/C | т/с | T/C | | T/C | |
| | S4-1 (ClearVue), S4-2 Compact (Affiniti), S5-1 (Affiniti, EPIQ), S8-3 Compact (Affiniti, EPIQ), S12-4 Compact (Affiniti, EPIQ) | | т/с | т/с | т/с | | т/с | |
| Phillips | V6-2 (Affiniti, ClearVue, EPIQ, HD5), VL13-5, VL13-5 Compact | | т/с | т/с | т/с | | T/C | |
| | X5-1 (EPIQ), X6-1 | | T/C | T/C | T/C | | T/C | |
| | 1.9MHz CW Pencil, 1.9MHz TCD | | T/C | т/с | T/C | | т/с | |
| | 5.0MHz CW Pencil | | T/C | T/C | T/C | | T/C | |
| | D2cwc, D2tcd, D5cwc | | T/C | T/C | T/C | | T/C | |
| | TCD | | T/C | T/C | T/C | | T/C | |
| | S6-2mpt S7-3t, S7-2t Omni, S8-3t | | С | С | С | | т/с/н | |
| | OMNI III TEE | | С | С | С | | T/C/H | |
| | X7-2t | | С | С | С | | T/C/H | |
| | L10-4lap | | T/C | T/C | T/C | | T/C/H | |
| | 15-6L | | T/C | T/C | T/C | | С | |
| CO2 Transducer | M1460A | | | | | | | |
| | Avalon Fetal Transducer | | | | | | | |

- **C** = Cable, connector, and strain reliefs
- **H** = Handle and steering mechanism
- **T** = Transducer and flexible shaft







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|------------------------------|--|--|---|--|--|---------------------------------|---|---|
| | und/Transducer bes (Other) | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch® Hospital Cleaner Disinfectant Towels with Bleach | Clorox® Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | ULTRASOUND TRANSDUCERS & PROBES (SYSTEM NAMES): | | | | | | | |
| | Acuson CV70 , Acuson SC2000 , Acuson Cypress , Acuson X500 , Acuson X300, Acuson Antares , Acuson X150, and Sonoline G20 UltraSound Systems | | | | | | | |
| | 3V2c, 4V1c, 5.0L45, 5.0P10, 6C1HD, 7.5L75S, 8V3, 10V4 | | | | | | | |
| | BE9-4 | | | | | | | |
| Siemens | C6-2, C7F2, C8-5, CW2, CW5, C5-2, C5F1, CH4-1, CH6-2, CX5-2 | | | | | | | |
| | EC9-4, EV9-4 | | | | | | | |
| | L10-5 | | | | | | | |
| | MPT7-4 | | | | | | | |
| | P4-2, P9-4, P10-4, PH4-1, PX4-1 | | | | | | | |
| | AUX CW | | | | | | | |
| | VF13-5-SP, VF7-3, VFX13-5, V5Ms, VF13-5-SP, V7M | | | | | | | |
| | SYSTEMS | | | | | | | |
| FUJIFILM | Edge™ | | | | | | | |
| SONOSITE, INC. www.sonosite. | Edge II | | | | | | | |
| com/support/ | FC1 | | | | | | | |
| cleaners- disinfectants | SII | | | | | | | |
| | X-Porte | | | | | | | |







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|---|---|--|---|--|--|---------------------------------|--|---|
| | ınd/Transducer pes (Other) | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | Wipes | |
| Handiacturer | TRANSDUCERS | | | | | | | |
| FUJIFILM SONOSITE, INC. www.sonosite. com/support/ cleaners- disinfectants | C11n, C11x, C11xp, C60n, C60x, C60xf, C60xf, C60xp, C8x, D2xp, HFL38xs, HFL38xi, HFL38xp, HFL50xp, HSL25x, HSL25xp, ICTn, ICTx, ICTxp, L25n, L25x, L25xp, L38n, L38x, L38x, L38xp, L52n, L52x, L52x Armored, P10x, P10xp, P21n, P21x, P21xp, rC60xi, rP19x, rP19xp, SLAx C60v, L25v, P21v, ACCESSORIES Battery Power Pack, External Power Supply, Power Supply, | | | | | | | |
| | Power Pack, TTC-Legacy, X-Porte Stand | | | | | | | |
| | Dock, FC1 Stand, H-Universal Stand, SII Stand, TTC-Edge, TTC_S Series, V-Universal Stand | | | | | | | |
| | ECG Cable, Mini Dock, | | | | | | | |
| | Edge II Stand, Edge Stand | | | | | | | |
| Welch Allyn | SureTemp® Therms and Probes | | | | | | | |
| | Transducers | | | | | | | |
| Toshiba | Diagnostic Ultrasound System Aplio i900, i800 and i700 | | | | | | | |
| Echosens | FibroScan 502/530 Touch Ultrasound and Probes | | | | | | | |







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|------------------------------|---|--|---|--|--|---------------------------------|---|---|
| Keybo | ards/Phones/ Cases | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| Whitney Medical Solutions | eShield* Screen Protectors and Cases | | | | | | | |
| KeySource International | San-a-Key® Keyboards | | | | | | | |
| Man and Machine | Keyboards and Mice | | | | | | | |
| Otterbox | Phone and iPad® cases | | | | | | | |
| Seal Shield | Medical Grade Keyboards, Mice, and Screen Protectors | | | | | | | |







| | | | | Hydrogen Peroxide | Quat | | | |
|--------------------------|--|--|---|--|---|---------------------------------|---|---|
| | Lights | Clorox Healthcare* Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch® Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| Skytron | Precision HD Handle Camera System | | | | | | | |
| | OR Lights | | | | | | | |
| | Exam Lighting | | | | | | | |
| Welch Allyn | Green Series™ 300 General Exam Lights | | | | | | | |
| | ProXenon 350 Surgical Illuminator | | | | | 1:10 | | |
| Trumpf | iLED 5/TruLight 5000 Lighting System | | | | | | | |
| Sunoptic Technologies | Surgical S400T Titan 400HP/ TITAN 180 Xenon Illuminator | | | | | | | |







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|-------------------------------|--|--|---|--|--|---------------------------------|---|---|
| | Sign Monitors/ Consoles | Clorox Healthcare* Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| Barco | Eonis® Diagnostic Displays | | | | | | | |
| | Transport Pro | | | | | | | |
| GE Healthcare | ApexPro Telemetry System | | | | | 1:10-1:100 | | |
| | ApexPro Antenna Infrastructure, Transmitter and Receiver | | | | | 1:10-1:100 | | |
| Midmark | IQvitals | | | | | | | |
| Mullark | IQvitals* Blood Pressure Cuffs | | | | | | | |
| | IntelliVue Patient Mx400/450/500/ 600/700/800 | | | | | | | |
| Phillips | IntelliVue Telemetry Transmitters | | | | | | | |
| | IntelliVue Monitors | | | | | | | |
| | Avalon Fetal Monitors | | | | | | | |
| | Connex Vital Signs | | | | | | | |
| | Connex Integrated Wall System | | | | | 1:10 | | |
| | Accessory Cable Management Mobile Stand 4800/4900 Series | | | | | | | |
| | CareTemp Touch Free Thermometer | | | | | 1:10 | | |
| Welch Allyn | Propaq Encore® Blood Pressure Cuffs | | | | | | | |
| | Harvey™ Elite® Stethoscopes | | | | | | | |
| | LumiView™ Binocular Microscope | | | | | | | |
| | Digital MacroView™ Oto- scopes and Halogen HPX® Illuminators | | | | | | | |
| | Accutorr 7 Vital Signs Monitor | | | | | | | |
| MindRay | Passport 12/Passport 8 Patient Monitor | | | | | | | |
| Mortara Instru- ment, Inc. | ELI 280/150c/250c 12-lead Resting Electrocardiograph | | | | | 1:500 to 1:10 | | |
| Capsule Tech | SmartLinx Vitals Plus | | | | | 10% | | |
| Exergen | TAT5000 Temporal Thermometer | | | | | | | |







| | | | | Hydrogen Peroxide | Quat | | | |
|-----------------|--|--|---|--|---|---------------------------------|---|---|
| Ra | adiology | Clorox Healthcare* Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch® Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | Computed Radiology Cassettes | | | | | | | |
| AGFA Healthcare | DR Detectors (Dx-D 10C, 10G, 20C, 20G) | | | | | | | |
| | DX-D 100 Digital Radiography X-Ray System | | | | | | | |
| Hologic | GemStar® Infusion Pumps | | | | | | | |
| InFab | Aprons and Gloves | | | | | | | |
| GE Healthcare | Optima XR220amx X-Ray System | | | | | 5250-6150 ppm | | |
| | Senographe Pristina Mammography System | | | | | | | |
| Konica Minolta | CS-1 Regius | | | | | | | |







| | | | | Hydrogen Peroxide | Quat | | | |
|----------------|---|--|---|--|---|---------------------------------|---|---|
| Ventil | ator Systems | Clorox Healthcare* Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch® Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| | Puritan Bennett | | | | | | | |
| | 540 series (409 was OK'd for touchscreen and arm) | | | | | | | |
| Covidien | 840 series (409 was OK'd for touchscreen and arm) | | | | | | | |
| | 980 series (409 was OK'd for touchscreen and arm) | | | | | | | |
| Kendall SCD | 700 Sequential Compression System | | | | | | | |
| AV Impluse | 6060 Controller and Accessories | | | | | | | |
| | Triology 100 System | | | | | | | |
| Phillips | Respironics V60/Plus Ventilato | | | | | 5%; 1:10 | | |
| Getinge-Maquet | SERVO i/s Ventilator System and Tube | | | | | | | |







| | | | | Bleach | | | Hydrogen Peroxide | Quat |
|--|---|--|---|--|--|---------------------------------|---|---|
| He | althcare IT | Clorox Healthcare® Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare* Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Manufacturer | Product | | | | | | | |
| Enovate Medical | Computer Carts (EMC) | | | | | | | |
| RICOH USA | Interactive Whiteboard | | | | | | | |
| Panasonic | TOUGHBOOK* 20, 33, 55, A3, G1, L1, N1, T1 | | | | | | | |
| GETT | Cleankeys® (wired and wireless), InduSense® | | | | | | | |
| | Mobile Cart Computing Solution Venus: 243/223/222/192 | | | | | | | |
| | Fanless Slim HIT Panel PC Solution ONYX- BE182/1723/1731/ 1531/1222/1022 | | | | | | | |
| Onyx Healthcare | Patient Monitor & Smart View Medical Solution ZEUS- 248/228/198 | | | | | | | |
| , | Medical All-In-One Solution AIO-MATE-2403/2203/1903 | | | | | | | |
| | Healthcare Infotainment Solution ONYX-BE321/BE182/1531/ 1222 MedPC- 2100/1600 | | | | | | | |
| | Mobile Medical Assistant MD116/MD101 | | | | | | | |
| | Medical Display Series Solution MEDDP- 524/522/519/518/ 517/515 | | | | | | | |
| НР | Healthcare Edition Notebook/ All in One | | | | | | | |
| Stanley Healthcare | Hugs and Kisses Tags | | | | | | | |
| GE Healthcare | Carescape Central Station MP100 | | | | | 5.25%; 1:10 dilution | | |
| TouchPoint InterMetro Industries Corp. | Metro 1800 Series Mobile Workstation | | | | | | | |
| Capsule Tech | SmartLinx Axon | | | | | 10% | | Clorox Disinfecting Wipes |
| | SmartLinx Device Identification Module | | | | | 10% | | |
| Howard Medical | E Series Computerized Work Station | | | | | | | |
| SealShield | Keyboards, mouse products, screen protectors | | | | | 1/2 cup to 1 gallon | | |





Clorox Healthcare® Equipment Compatibility Guide

| Manufacturer Product Blickman Surgical Steel Carefusion Pyxis* Supply System Roche Cobas* Urisys 1100* Urine Analyzers Sechrist Chambers RF Surgical Systems RF Assure Detection Medline BioCon 750 Bladder PhysioControl Defibrillator WelchAllyn Suresight* Vision Scr Autorefractor GE Healthcare Giraffe OmniBed Care CS1 Custom Ultrasonics 83-S6 Preprocessing Ultrasonics AirClean Systems Workstation Gentec 882VR Continuous S Regulators Tomey USA Pachymeter Probe LivaNova Sorin Group T Heater-Cooler Syst SeliteOxygen Concent NewLife EliteOxygen Concent NewLife EliteOxygen Concent Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Healthcare, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. Covaris CryoPREP Extraction | | | | Bleach | | | Hydrogen Peroxide | Quat |
|--|------------|--|---|---|--|---------------------------------|---|---|
| Blickman Surgical Steel Carefusion Pyxis* Supply System Roche Cobas* Urisys 1100* Urine Analyzers Sechrist Chambers RF Surgical Systems RF Assure Detection Medline BioCon 750 Bladder PhysioControl Defibrillator WelchAllyn Suresight* Vision Scr. Autorefractor GE Healthcare Giraffe OmniBed Care CS1 Custom Ultrasonics 83-S6 Preprocessing Ultrasonics AirClean AC600 Series Lamina Systems Workstation Gentec Regulators Tomey USA Pachymeter Probe LivaNova Sorin Group Dwyer Precision Products BreathCall Series Mo Chart Industries AirSep QuietLife 5 EliteOxygen Concent AirSep AirSep QuietLife 5 EliteOxygen Concent Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. Uries Supply Systems Viriant Supply Systems Uries Supply | | Clorox Healthcare* Bleach Germicidal Wipes | Clorox Healthcare* Bleach Germicidal Cleaners | Clorox Healthcare Fuzion Cleaner Disinfectant | Dispatch* Hospital Cleaner Disinfectant Towels with Bleach | Clorox* Germicidal Bleach | Clorox Healthcare* Hydrogen Peroxide Cleaner Disinfectant Wipes | Clorox Healthcare* VersaSure Alcohol-Free Disinfectant Wipes |
| Carefusion Pyxis* Supply System Roche Cobas* Urisys 1100* Urine Analyzers Sechrist Chambers RF Surgical Systems RF Assure Detection Medline BioCon 750 Bladder PhysioControl Defibrillator WelchAllyn Suresight* Vision Scr. Autorefractor GE Healthcare Giraffe OmniBed Care CS1 Custom Ultrasonics 83-S6 Preprocessing Ultrasonics Ac600 Series Lamina Systems Workstation Gentec Regulators Tomey USA Pachymeter Probe LivaNova Sorin Group Dwyer Precision Products AirSep AirSep QuietLife 5 EliteOxygen Concent AirSep AirSep QuietLife 5 EliteOxygen Concent Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. UL-EV100 Transposa Evacuation Unit | | | | | | | | |
| Roche Cobas* Urisys 1100* Urine Analyzers Sechrist Chambers RF Surgical Systems RF Assure Detection Medline BioCon 750 Bladder PhysioControl Defibrillator WelchAllyn Suresight* Vision Scr. Autorefractor GE Healthcare Giraffe OmniBed Care CS1 Custom Ultrasonics AirClean AC600 Series Lamina Systems Workstation Gentec Regulators Tomey USA Pachymeter Probe LivaNova Sorin Group Dwyer Precision Products AirSep QuietLife 5 EliteOxygen Concent AirSep AirSep QuietLife 5 EliteOxygen Concent Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. | | | | | | | | |
| Sechrist Chambers RF Surgical Systems RF Assure Detection Medline BioCon 750 Bladder PhysioControl Defibrillator WelchAllyn Suresight* Vision Scr Autorefractor GE Healthcare Giraffe OmniBed Care CS1 Custom Ultrasonics AirClean AC600 Series Lamina Systems Workstation Gentec Regulators Tomey USA Pachymeter Probe LivaNova Sorin Group Dwyer Precision Products AirSep QuietLife 5 EliteOxygen Concent AirSep AirSep QuietLife 5 EliteOxygen Concent AirSep NewLife EliteOxygen Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. BioCon 750 Bladder BioCon 750 Bladder BioCon 750 Bladder Autorefractor Autorefractor B3-S6 Preprocessing Workstation A600 Series Lamina Workstation T Heater-Cooler Syst EliteOxygen Concent Systems More Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators UL-EV100 Transposa Evacuation Unit | ns | | | | | | | |
| RF Surgical Systems Medline BioCon 750 Bladder PhysioControl Custom Ultrasonics AirClean Systems AirClean Systems Tomey USA LivaNova Sorin Group Dwyer Precision Products Chart Industries Germfree Germfree Besmax Health- care, Inc. EIFEPAK 15 Monitor/ Defibrillator Suresight* Vision Scr. Autorefractor Giraffe OmniBed Care CS1 Ac600 Series Lamina Workstation 882VR Continuous S Regulators T Heater-Cooler Syst BreathCall Series Mo Products AirSep QuietLife 5 EliteOxygen Concent Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector Zimmer Biomet Dornoch Medical Systems, Inc. LIFEPAK 15 Monitor BioCon 750 Bladder Autorefractor Ba3-S6 Preprocessing Workstation Ba2-VR Continuous S Regulators T Heater-Cooler Syst EliteOxygen Concent Systems, Inc. | | | | | | | | |
| Systems RF Assure Detection Medline BioCon 750 Bladder PhysioControl LIFEPAK 15 Monitor/ Defibrillator WelchAllyn Suresight* Vision Scr. Autorefractor GE Healthcare CS1 Custom Ultrasonics 83-S6 Preprocessing Workstation 882VR Continuous S. Regulators Tomey USA Pachymeter Probe LivaNova Sorin Group Theater-Cooler Syst Dwyer Precision Products AirSep QuietLife 5 EliteOxygen Concent AirSep Reprocessing Workstation BreathCall Series Mo Pachymeter Probe AirSep QuietLife 5 EliteOxygen Concent Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector Zimmer Biomet Dornoch Medical Systems, Inc. LIFEPAK 15 Monitor/ Defibrillator 8uresight* Vision Scr. Autorefractor 83-S6 Preprocessing Workstation 82VR Continuous S. Regulators T Heater-Cooler Syst EliteOxygen Concent NewLife EliteOxygen Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector Zimmer Biomet Dornoch Medical Systems, Inc. | | | | | | | | |
| PhysioControl Defibrillator WelchAllyn Suresight* Vision Scr. Autorefractor GE Healthcare Giraffe OmniBed Care CS1 Custom Ultrasonics 83-S6 Preprocessing Workstation Gentec 882VR Continuous S Regulators Tomey USA Pachymeter Probe LivaNova Sorin Group Dwyer Precision Products Chart Industries AirSep QuietLife 5 EliteOxygen Concent AirSep Germfree BUBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Healthcare, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. | System | | | | | | | |
| Physiocontrol Defibrillator WelchAllyn Suresight* Vision Scr. Autorefractor GE Healthcare CS1 Custom Ultrasonics AirClean Systems Workstation Gentec Beautors Tomey USA LivaNova Sorin Group Dwyer Precision Products Chart Industries AirSep Germfree Germfree By I Series By Series Laminatory By By | Scanner | | | | | | | |
| WelchAllyn Autorefractor GE Healthcare Giraffe OmniBed Care CS1 Custom Ultrasonics AirClean Systems Gentec Bestin Coup Dwyer Precision Products Chart Industries Germfree Germfree Germfree Germfree Besmax Healthcare, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. Giraffe OmniBed Care CS1 AC600 Series Lamina Workstation 882VR Continuous S Regulators Regulators T Heater-Cooler Syst BreathCall Series Mo AirSep QuietLife 5 EliteOxygen Concent NewLife EliteOxygen Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector UL-EV100 Transposa Evacuation Unit | , | | | | | | | |
| Custom Ultrasonics AirClean Systems AC600 Series Lamina Systems AC600 Series Lamina Workstation BazVR Continuous S Regulators Tomey USA LivaNova Sorin Group Dwyer Precision Products Chart Industries AirSep AirSep AirSep BreathCall Series Mo NewLife EliteOxygen Concent Laminar Flow Workst VersaFlow Compoun Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. AC600 Series Lamina SazVR Continuous S Regulators T Heater-Cooler Syst EliteOxygen Cooler Syst Series Mo AirSep QuietLife 5 EliteOxygen Concent NewLife EliteOxygen Concentrator BVBI Series/BZ Series Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector UL-EV100 Transposa Evacuation Unit | reener and | | | | | 1:10 | | |
| Ultrasonics AirClean Systems Ac600 Series Lamina Workstation BasavR Continuous S Regulators Tomey USA LivaNova Sorin Group Dwyer Precision Products Chart Industries AirSep Germfree BreathCall Series Mo NewLife EliteOxygen Concent Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector Zimmer Biomet Dornoch Medical Systems, Inc. Ac600 Series Lamina BasavR Gerulina Savar Realth Care, Inc. Ac600 Series Lamina Savar Regulators Theater-Cooler Syst Fletter-Cooler Syst Fletter-Cooler Syst Series Mo Products AirSep QuietLife 5 EliteOxygen Concent Concentrator BVBI Series/BZ Series Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector UL-EV100 Transposa Evacuation Unit | estation | | | | | 0.50% | | |
| Systems Workstation Gentec 882VR Continuous S Regulators Tomey USA Pachymeter Probe LivaNova Sorin Group T Heater-Cooler Syst Dwyer Precision Products AirSep QuietLife 5 EliteOxygen Concent AirSep NewLife EliteOxygen Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Health- care, Inc. Curaplex Oxygen Nip Connector Zimmer Biomet Dornoch Medical Systems, Inc. | g Sink | | | | | 1:32 | | |
| Tomey USA Pachymeter Probe LivaNova Sorin Group T Heater-Cooler Syst Dwyer Precision Products Chart Industries AirSep Germfree Germfree Germfree Besmax Healthcare, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. Pachymeter Probe AirSep QuietLife 5 EliteOxygen Concent NewLife EliteOxygen Concentrator BVBI Series/BZ Series Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector UL-EV100 Transposa Evacuation Unit | ar Flow | | | | | 1:10 | | |
| LivaNova Sorin Group Dwyer Precision Products Chart Industries AirSep AirSep AirSep MewLife EliteOxygen Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. Theater-Cooler Syst BreathCall Series Mo AirSep QuietLife 5 EliteOxygen Concent EliteOxygen Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators UL-EV100 Transposa Evacuation Unit | Suction | | | | | | | |
| Group Dwyer Precision Products Chart Industries AirSep AirSep MewLife EliteOxygen Concent Concentrator Germfree Germfree BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Healthcare, Inc. Cimmer Biomet Dornoch Medical Systems, Inc. BreathCall Series Mo NewLife EliteOxygen Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators UL-EV100 Transposa Evacuation Unit | | | | | | | | |
| Products Chart Industries AirSep QuietLife 5 EliteOxygen Concent NewLife EliteOxygen Concentrator BVBI Series/BZ Serie Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. BreathCall Series Mo AirSep QuietLife 5 EliteOxygen Concent VersaFlow Compoun Connector UL-EV100 Transposa Evacuation Unit | tem | | | | | | | |
| AirSep Bermfree Germfree Germfree Germfree BVBI Series/BZ Series Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector Zimmer Biomet Dornoch Medical Systems, Inc. EliteOxygen Concent | unting | | | | | | E | Broad Spectrum |
| Germfree Germfree BVBI Series/BZ Series Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector Zimmer Biomet Dornoch Medical Systems, Inc. Uncertain UL-EV100 Transposa Evacuation Unit | trator | | | | | | | |
| Germfree Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. Laminar Flow Workst VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector UL-EV100 Transposa Evacuation Unit | | | | | | | | |
| Besmax Health- care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. VersaFlow Compoun Aseptic Isolators Curaplex Oxygen Nip Connector UL-EV100 Transposa Evacuation Unit | station | | | | | 5% | | |
| care, Inc. Zimmer Biomet Dornoch Medical Systems, Inc. Connector UL-EV100 Transposa Evacuation Unit | | | | | | <1000 ppm | | |
| Dornoch Medical Systems, Inc. UL-EVIOO Transposa Evacuation Unit | ople | | | | | 0.5% | | |
| Covaris cryoPREP Extraction | ıl Ultra | | | | | <1200 ppm | | |
| | ı System | | | | | 10% | | |
| Fresenius 2008K series | | | | | | | | |
| Medical Care 20081 series | 11.11.5 | | | | | | | |
| Granuflo Concentrate | e Unit 1 | | | | | | | |
| Draeger Perseus A500 seris Nanosonics Trophon - chamber, e | ovtorio: | | | | | | | |